

Now, choose the heat and moisture exchange system right for your breathing needs and activity levels



Introducing the new
Blom-Singer®
EasyFlow® HME

Blom-Singer®
ClassicFlow® HME

EasyFlow® HME



ClassicFlow® HME



INHEALTH
TECHNOLOGIES®

Choosing the right HME is easy: Blom-Singer® HMEs

Take your pick: the Blom-Singer ClassicFlow or the new Blom-Singer EasyFlow. Both help ensure that you breathe humidified, filtered, and warmed air. Research shows that long-term HME use results in:


- *Less mucus*¹
- *Less coughing*¹
- *Less risk of shortness of breath, fatigue, sleep problems, anxiety, and depression*^{1,2}



The new Blom-Singer® EasyFlow® HME

Breathe easier at work or play

With EasyFlow, you can breathe a little easier. Its foam filter is more porous than the ClassicFlow. This allows you to breathe more freely to match your activity level and pulmonary needs.

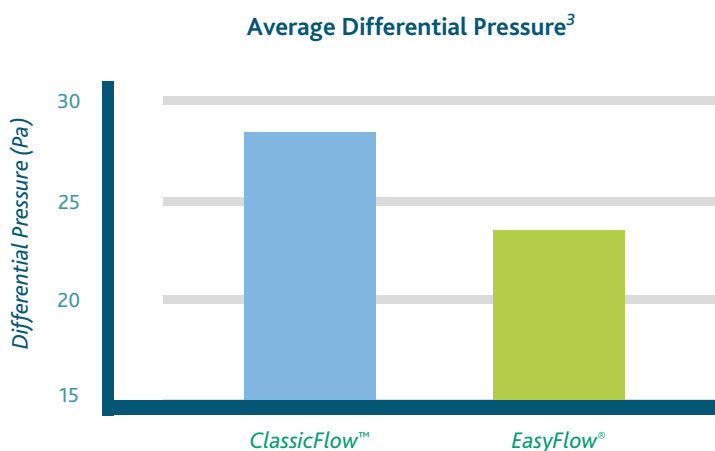

HME Cassette	Airflow Resistance ³	Moisture Exchange ³	Hygroscopic Antimicrobial Filter	24-Hour Use	EasyTouch Push to Speak	MucusShield Protection
EasyFlow®	Lower	Good	✓	✓	✓	✓
ClassicFlow®	Low	Better	✓	✓	✓	✓

Plus, EasyFlow's beige tint and non-reflective surface texture is less noticeable.

The Blom-Singer® ClassicFlow® HME

Breathe healthy day and night

ClassicFlow helps assure you breathe warmed, filtered, and humidified air—all day, every day—to reduce respiratory symptoms after a total laryngectomy.¹ ClassicFlow is designed for increased moisture retention and resistance.



Both share unique advantages

MucusShield™ helps protect the foam filter

With other HMEs, if mucus gets on the filter, the efficiency of the filter is reduced. Not so with Blom-Singer HMEs. The MucusShield technology guards the foam filter from direct contact with mucus to help enhance efficiency.

EasyTouch™ speech button makes speaking easier

When you want to talk, press the EasyTouch speech button to seal the valve and speak easily. Upon release, the EasyTouch returns to its open, breathing position.

24/7 heat and moisture exchange

Our HMEs are "daily-disposable," 24-hour-use devices. You simply put on a new one each day, and the benefits continue through the night.



Also available

Blom-Singer® HumidiFilter™ System

HumidiFilter is the simple, cost-effective alternative for heat and moisture exchange. Simply place a new filter once a day into the holder to get the benefits of wearing an HME. The holder is reusable and designed for daytime use.

Blom-Singer®

The voice of experience since 1978

For everyday or more active use, the choice is easy: Blom-Singer® HMEs

For more information or to order the new Blom-Singer® EasyFlow® HME, Blom-Singer® ClassicFlow® HME, or Blom-Singer® HumidiFilter™ System, contact InHealth Technologies.

Call **800.477.5969**

Email order@inhealth.com

Visit or order online www.inhealth.com

REF	DESCRIPTION
BE 1055EF	Blom-Singer EasyFlow HME — for 24-hour use, 30 per pack
BE 1055EZ	Blom-Singer ClassicFlow HME — for 24-hour use, 30 per pack
BE 1060	Blom-Singer HumidiFilter Holder, 1 each
BE 1080	Blom-Singer HumidiFilter Holder Foam Filters, 30 per pack

INHEALTH TECHNOLOGIES®

UNITED STATES

1110 Mark Avenue
Carpinteria, CA 93013-2918 USA
Toll Free: 800.477.5969
Toll Free Fax: 888.371.1530

ORDER ONLINE

www.inhealth.com

1. Ackerstaff AH, Hilgers FJ, et al. Improvements in respiratory and psychosocial functioning following total laryngectomy by the use of a heat and moisture exchanger. *Annals of Otolaryngology, Rhinology & Laryngology*. 1993;102(11):878-883.

2. Ackerstaff AH, Hilgers FJ, et al. Heat and moisture exchangers as a treatment option in the post-operative rehabilitation of laryngectomized patients. *Clinical Otolaryngology*. 1995;20:504-509.

3. Research and Product Development Study, HME Test. RD-237.